MD

# LUXIFLEX HAL

**MIMSAL**

## LUXIFLEX HALÓGENA

cód. 10104AO

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The design of the light fixture is highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

### 1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

### 2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

### 3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.
- **To change the bulb:**
  - Always disconnect the lamp from the power.
  - Remove the ring cover from the lampshade (*image 1 and 2*).
  - Remove the bulb and replace the new bulb.
  - Replace the ring cover (*image 2 and 1*).

Notify by the user / patient to MIMSAL and the competent authority of the country, any serious incident that occurs in relation to the use of the equipment

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	LUXIFLEX HALOGEN
Light bulb	Hal. Masterline ES 35W 8°
Useful life	5.000 h
Lux at 50 cm	50.000 lux
Colour temperature	4.000° K
Colour Rendering Index	95%
Electrical Protection Class	Class II
Total Length	85 cm
Color	White
Protective glass	Borosilicate
Power	220/230V
Frequency	50/60 Hz

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