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
## Instructions for LED nystagmus spectacles

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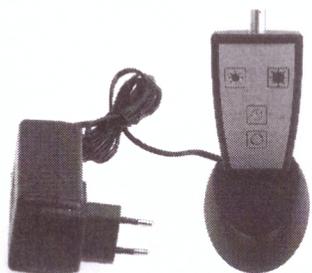Rev. 2017.06

en

### General informations/Safety informations

Read these instructions carefully and keep them in a safe place for future reference before using the product. Misuse or any modification is not under the responsibility of the manufacturer. Please check every instrument before every use. FAROMED GmbH strictly forbids any modifications of the instruments. The CE-sign looses its validity in case of the use of any non original spare parts. This product may only be used by medical staff. These instructions do not replace a good medical education.

### Intended use

Instrument intended for short-term and mobile examination in the area of vertigo diagnostic.

### Warnings

Do not use damaged instruments! Protect the spectacles and its parts against all kind of mechanical damage or force. Do not throw and use with care. Screen the headlight and the accessories from the sun. Keep dry! The products have to shelter from humidity about 75 %!

Misuse, damage, and wear are not covered by the warranty. Any signs of wear and tear due to regular use are excluded from the warranty.

Make absolutely sure that no liquid enters the battery handle or the spectacle. Before any use the product has to be cleaned/ disinfected and proper functionality and integrity has to be verified. Contact your dealer or the manufacturer for repairs or in any case of damage.

### Warnings – Switch housing incl. Li-Ion rechargeable battery

- Use the enclosed charger with tray only.
- Do not heat or throw the switch housing of into fire.
- Do not charge and leave the switch housing at the high temperature.
- Do not deform, short-circuit, disassemble or modify the switch housing.
- Do not allow the switch housing to be immersed in or wetted with water or sea water.
- Do not subject the switch housing to mechanical load, a strong impact or a throw.
- Do not carry or store the switch housing together with material which have sharp edges or is electrical conductive in the same custody.
- Do not cut, squeeze and tear at the plug of the switch housing.
- Not letting the golden terminals for charging come in contact with electrical conductive or between them (short-circuit).
- Disregarding the above items may cause heat, fire and explosion!

### Warnings – Charger with tray

- The charging tray is designed for use with switch housing only.
- Charger is designed for indoor use only and should not come into contact with water, dust, oil, grease etc.
- The product should not be covered while in use.
- The main socket should be easily accessible.
- In the event of operational error, the plug should be immediately removed from the socket.
- Do not open the wall plug charger and the charging tray.
- The product should not be used in the vicinity of flammable anaesthesia gases.
- The charger has an internal fuse which blows if a fault occurs in the charger. In addition the charger is equipped with a fuse switch which cuts off the unit in the case of reverse polarity connection on the battery. Such faults must be repaired by qualified service personnel only.

### Recharging

1. At first connect the plug-in charger to the power socket.
  2. Deploy the charging tray on a table.
  3. Now put the switch housing into the charging tray. The charging is signalled by a red light on the plug-in charger.
- In the case of deeply discharged batteries, the charging indicator changes from green to red later.

en

4. The charge is completed if the LED-indicator changes to green. The Charging period will take approx. 4 hours.

### Instructions for use

Please recharge the switch housing completely before the first use. Connect the nystagmus spectacles with the switch housing.

**To connect the plug to the switch housing and to remove please handle at arrow-marked (←→) piece only! Do not turn the plug!**

If you do not use for a long time, disconnect the glasses from the switch housing, otherwise the batteries will be completely discharged. Recharge exhausted batteries!

### Switching on

To turn the spectacles on or off and to adjust the brightness push the respective button on the switch housing.

### Timer

To start the timer press the button indicated by the stopwatch symbol. The three LED on the spectacles represent 30, 60, and 90 seconds.

### Automatic switch-off / switch-on

One minute after the timer runs out the light will turn off automatically to conserve energy. The light will turn on again when the spectacles are picked up.

### Deactivating the automatic switch-off

Turn on the spectacles while pressing the button for the timer. The LED "30 sec" will lit for about 1 second and then go black. Now turn the spectacles off and on again. The automatic switch-off function is deactivated – indicated by a briefly lit LED "30 sec".

To reactivate the automatic switch-off function, repeat the same steps.

### Cleaning

Right after any contamination it is advised to clean the product immediately by using a soft and humid tissue and wiping all parts. Do not clean it by machine. Do not disinfect by spray or soak!

### Storage and transport conditions

Storage temperature	-30 to 60 °C
Working temperature	0 to 25 °C
Humidity	30 to 75 %
Barometric pressure	700 to 1,060 hPa

### Disposing

After the lifetime the instrument has to be littered in compliance with the local regulations and law. The following applies for the Europe-an Union: According to the EU Directive 2002/96/EC on Waste Electronic and Electrical Equipment (WEEE) do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by the directive. If this product is contaminated, this directive does not apply. Low batteries have to be removed and littered according to the local regulations and law.

### Technical specification

Total weight:	170 g (without switch housing and charger)
Classification:	Class I, according to annex IX of 93/42/EEC
Classification:	Risk group 1 according DIN EN 62471
Power supply:	Li-Ion rechargeable battery
Protection class:	IP X0
Charging period: approx.	4 h at fully discharge

### Spare parts and accessories

HF 801	Goggles with cable and plug
8.0121.01	Switch housing with Li-Ion rechargeable battery
8.0121.02	Charger with Charging tray for 08-434