


STAIRS WHEEL CHAIR code SED001


Instructions and Maintenance

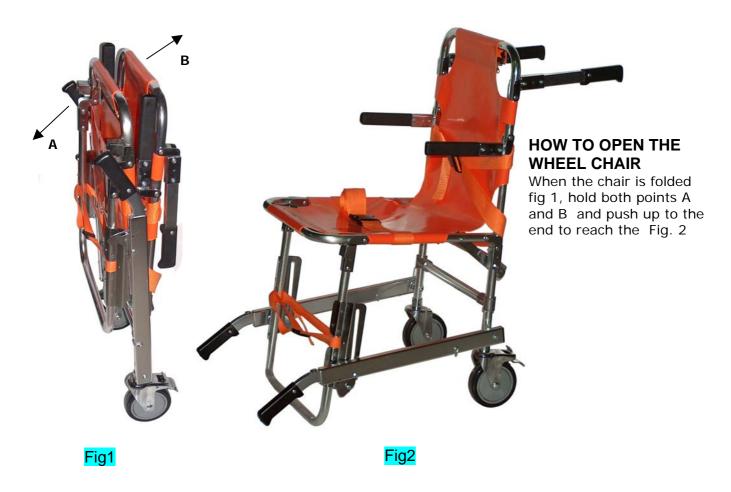

HOW TO PREPARE THE PATIENT TRANSPORTATION

Once the Patient is well seated, lower both the armchair pressurising the end higher part of the plastic protection, as in fig. 3. To bring the back armchair to an horizontal position pull them up.


To pull the front armchair down push and stop when they do click on the lock fig. 4
The lift bars can be lengthen pushing on the button and pulling on the handle towards you. You will get the wheel chair as in fig. 2.

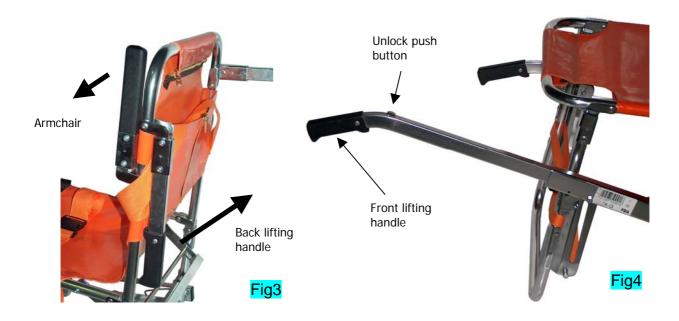

SECURITY SYSTEMS

To have a better stability when the chair is open, the left wheel can be locked, fig. 5, pressing the foot on the A point. To unlock press once again with your foot on the B

To have the front lift bar at the same position as when the wheel chair is folded, hold the lock as in fig. 6, pull it towards you, freeing the pivot then lift the handle. Quick locked belts are supplied with the wheel chair for the Patient security

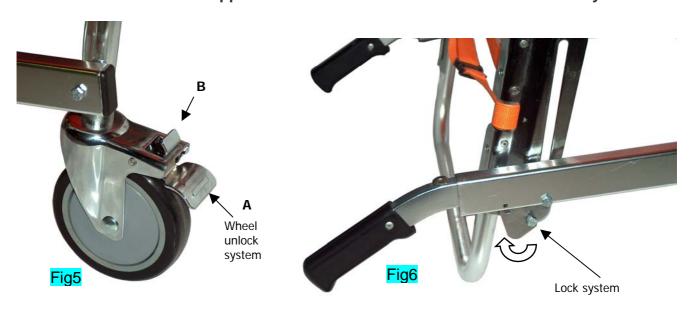



HOW TO FOLD THE WHEEL CHAIR

Bring back all the handles to the initial positions. With one hand pull up the bar as in fig. 7, with the second hand pull the wheel chair aluminium profile. Pull in as much as possible and the wheel chair will be fully folded as in

For a better security the wheel chair is supplied with a belt which stops any accidental opening

HOW TO KEEP THE WHEEL CHAIR **CLEAN AND TO STORE THEM**

Using equipment disinfectant solutions. Medical devices . Store them in a clean and dry room far away from direct heat.

Sizes:

Folded: mm.910H x 195P x 520L : mm.910H x 710P x 520L Open

Weight: 8,50 Kg. Max load: 150 Kg.

