MultiEquipment. Ref F336

Manual de instrucciones

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MultiEquipment. Ref F336

Instruction manual

Diamond and microcrystal dermabrasion

Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

weelko
the wellness
company

Importador/Importer
Importateur/Importeur
Importatore:

Weelko Barcelona, S.L.
ESB65397556

Barcelona
Pol. Ind. La Vall dan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 938 24 90 20
E. info@weelko.com
W. www.weelko.com

Front panel


Instructions of use

1. Connect the power supply via the rear panel.
2. Turn on the "POWER" button.
3. Press "FUNCTION" to select the desired function in the following order: micro-crystal dermabrasion, diamond dermabrasion.

MICRO-CRYSTAL DERMABRASION


Micro-crystal containers: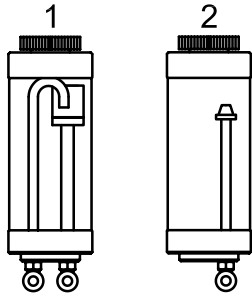


1. Used micro-crystal container.
2. New micro-crystal container.

Instructions of use

1. Assemble the plastic handle holder in one side of the device.
2. Properly plug the pipes as shown in the sketch below. Match each letter on the pipes with each letter on the containers. Just introduce them and do gentle pressure. To withdraw the pipes, press the round plastic part in the plug and pull the pipe without forcing.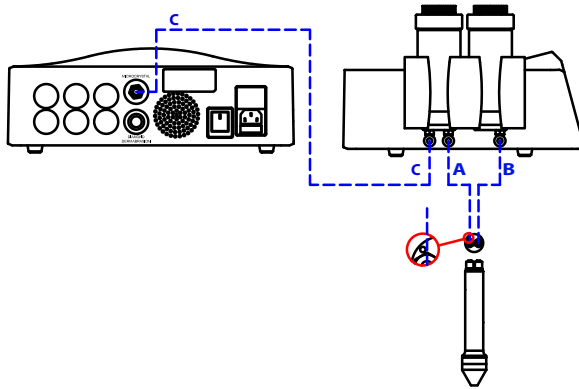

3. Fill with micro-crystals the container #2 without exceeding the line. The cap thread must be completely clean of micro-crystals. #120 particle size micro-crystals are recommended to be used.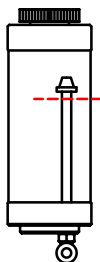

Important note:

- Please check that the head of the handle is not blocking the hole inside.
 - Do not manipulate micro-crystals with hands.
4. Select the desired power output and time of working by pressing “ENERGY” and “TIMER” respectively. “ENERGY” can be adjusted between 1 and 10. “TIMER” can be adjusted between 1 and 60 minutes.
 5. Press “START/PAUSE” button to start. The display indicates the machine is working normally. The operation can be paused and restarted by pressing this button.
 6. Once the operation time is finished, the device will beep and the selected function will stop. The device is now in standby mode.
 7. Once the treatment is finished, micro-crystals are deposited in the container #1. On the contrary, container #2 is empty. Remove and discard the used micro-crystals.
- Fill the container #2 for future treatments. The cap thread must be completely clean of micro-crystals.

Notes

- During the treatment, always keep the handle under the device level. Do not work with the handle over the device level.
- The direct contact to the skin is 3–5 seconds only. After this time, lift the handle and apply it to another part of the treatment area.

Cautions


- Clean and dry skin. No need to apply lotion for treatment.
- During the operation on the eyes, please adjust the air pressure about 1-5 , and it will be enough after gently hand over twice.
- Avoid sun exposure for 24 hours after treatment.
- Do not use makeup for 4 hours after treatment.
- Do not make sauna sessions during the 10 days after treatment.
- After treatment, use neutral products to moisturize the skin rather than acidic.

DIAMOND DERMABRASION

1. Correctly connect one side of the pipe to the rear panel as indicated in the sketch.
2. Connect the opposite side of the pipe to the vacuum handle. Place the correct diamond head in the handle. There are nine types of diamond heads.

| Ø 22 MM | |
|------------|---|
| D100 | Body. Suitable for acne, knees, elbows and areas with stretch marks |
| D180 | |
| Ø 19 MM | |
| D180 | Face. Suitable for large spots, pores, aging sun exposure |
| D220 | |
| Ø 15 MM | |
| D100 | Scars, fine lines and various imperfections |
| D200 | |
| D250 | |
| Ø 8 Y 6 MM | |
| D200 | Eyes and both sides of the nose. |
| D250 | Wrinkles, acne, pimples and blackheads |

3. Check that the foam filter is in its proper position in the handle.
4. Check that the metal filter is in its proper position in the handle.

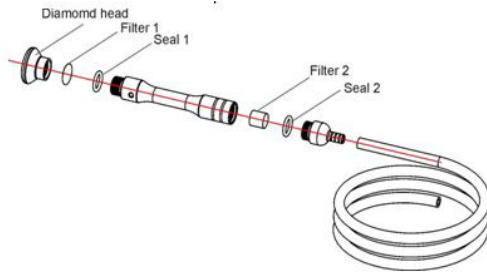5. You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.
6. Select the desired energy and time. You can change setting anytime during working.
7. Press "START/PAUSE" button to start.

Cautions

- Do not leave the handle on the same part of skin for too long since it can cause injuries.
- Avoid using it on irritated skin.
- The following skin areas are unsuitable for these treatments: skin tumors, moles, birthmarks, angiomas, lips, eyes, herpes and warts.
- Do not operate the suction pump for prolonged periods without cooling time between treatments to prevent pump overheating and damage to the machine. Damage caused by overheating of the pump are not covered by the manufacturer's warranty.

FAQ

Diamond dermabrasion

If the suction effect is reduced:

- Replace the metal and foam filters.

MAINTENANCE

Micro-crystal dermabrasion

- The filter in container #1 prevents micro crystals entering the suction circuit of the equipment. 4 replacement filters are delivered. To replace the filter, unscrew the cap of the container stand, remove the old filter, insert the new filter and screw the cap holder.
- Clean and disinfect the handle and after each treatment. It is recommended to dip first ultrasonic sterilizer and subsequently in an ultraviolet sterilizer.
- Do not use the appliance if the filter tube attached to the device is not installed.
- This filter should be replaced every 5 treatments.
- To replace the filter:
 - Unscrew the top of the tank.
 - Remove the filter by pulling and insert the new one.
 - Screw the top of the tank, making sure the silicone gasket is positioned correctly.

Note: Before screwing the top, wipe any traces of micro crystals in the thread and the silicone gasket to keep a proper sealing.

4 replacement filters are delivered for their replacement.

Diamond dermabrasion

- The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- The metal filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- Clean and sterilize the diamond heads after every treatment.

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