ES

F834
Manual de instrucciones
WE-2019-F834

MICRODERMOABRASIÓN

ADVERTENCIA

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MICRODERMABRASION

WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU

2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to

incorrect operation.




The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS


• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	TUBES	10029	1
2	HANDLE	-	2
3	BAG OF FILTERS AND SPARES	40010	1
4	DIAMOND HEAD Ø6		1
5	DIAMOND HEAD Ø8		1
6	DIAMOND HEAD Ø15		3
7	DIAMOND HEAD Ø19		2
8	DIAMOND HEAD Ø22		2
9	MAIN UNIT	-	1
10	FEEDER CABLE	-	1

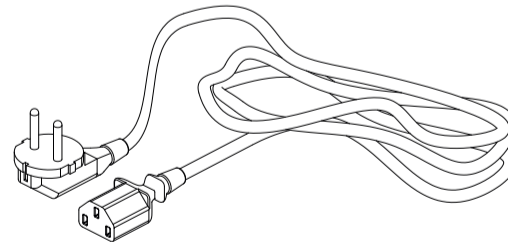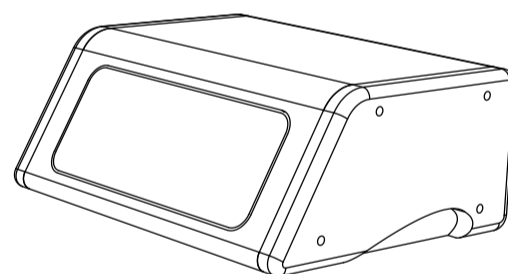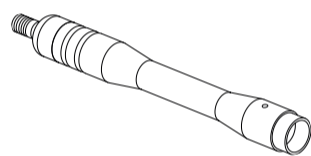FRONT PANEL/REAR PANEL

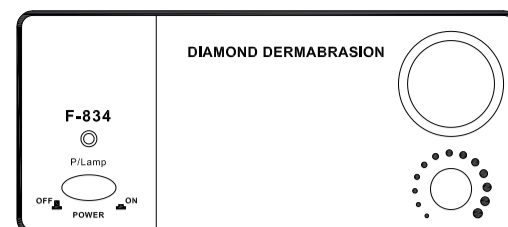
1. Tube

2. Power line input


INSTRUCTIONS OF USE

1. Connect the power supply via the rear panel, indicated in the sketch as "POWER LINE INPUT".

2. Correctly connect one side of the pipe to the rear panel as indicated in the sketch.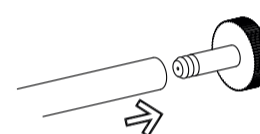

3. Connect the opposite side of the pipe to the vacuum handle. Place the correct diamond head in the handle. There are nine types of diamond heads.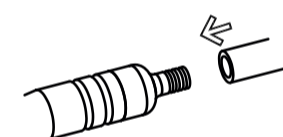

Ø 22 MM

D100 Body. Suitable for acne, knees, elbows and areas with stretch marks
D180

Ø 19 MM

D180 Face. Suitable for large spots, pores, aging sun exposure
D220


Ø 15 MM

D100 Scars, fine lines and various imperfections
D200


Ø 8 AND 6 MM

D200 Eyes and both sides of the nose. Wrinkles, acné, pimples and blackheads
D250

4. Check that the foam filter is in its proper position in the handle.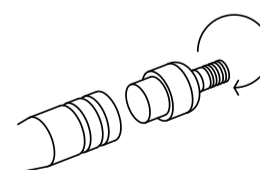

5. Check that the metal filter is in its proper position in the handle.

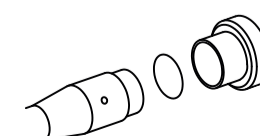6. You can adjust the suction effect by turning the regulator.

7. You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.

8. Once you have finished, first turn off the unit by pressing the "POWER" button.

CAUTIONS

• Do not leave the handle on the same part of skin for too long since it can cause injuries.

• Avoid using it on irritated skin.

• The following skin areas are unsuitable for these treatments: skin tumors, moles, birthmarks, angiomas, lips, eyes, herpes and warts.

• The pressure regulator can be unscrewed completely. If this happens accidentally, you can screw it back on and the machine will still work correctly.

• Do not work the vacuum pump for extended periods without cool-down time between treatments in order to prevent overheating of the pump and damage to the machine. Damage caused by overheating of the pump is not covered by the manufacturer's warranty.

FAQ

If the unit does not suck under the action of the pressure regulator:

• Unscrew the pressure regulator completely.

• Check that the black rubber valve on the end of the regulator is not broken.

• If it is, remove the valve pieces of both the unit and the regulator.

• Take one of the black rubber valves that are supplied and insert it in the regulator, exerting pressure.

If the suction effect is reduced:

• Replace the metal and foam filters.

MAINTENANCE

• The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.

• The metal filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.

• Clean and sterilize the diamond heads after every treatment.

ACCESSORIES

REF.	DESCRIPTION
10029	TUBES
40010	BAG OF FILTERS AND SPARES
	DIAMOND HEAD Ø6
	DIAMOND HEAD Ø8
	DIAMOND HEAD Ø15
	DIAMOND HEAD Ø19
	DIAMOND HEAD Ø22

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	36x31x1,85
	Gross weight (kg)	3,8
	Product size (cm)	31x17/24x13
Working characteristics	Working power	65 W
	Working pressure	0 bar ~ -0,8 bar
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	65 W

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