ES Manual de instrucciones  
EN Instruction manual  
FR Mode d'emploi  
DE Bedienungsanleitung  
IT Manuale d'istruzioni

**WKE007**

## **EPILCAN**

CALENTADOR DE CERA  
WAX HEATER  
CHAUFFE-CIRE  
WACHSERHITZER  
SCALDACERA

## USO DEL MANUAL

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## USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

---

## WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

---

## EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

## RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

---

## WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## CONTENT

On arrival of the package, check the presence of the following items:

- Wax heater

## OPERATION

1. Put the wax into the inner housing and connect the machine to the power.
2. Please read this document carefully before using the device. Keep it in an accessible place so that you can read it at any time.
3. When the indicator light turns into green, the machine starts heating.
4. Then, adjust the temperature accordingly to the patient comfort.
5. Before starting to remove the hair, clean the grease and cosmetic that may remain on the skin with cotton that has been previously soaked in an astringent lotion. You can also use a toilet powder that absorbs moisture and grease. This will make the hair adhere to the wax more easily.
6. Daub the soft wax following the direction the hair growth.
7. After being heated, the wax should be applied with a wooden spatula on the skin to be treated. The thinner the wax layer is, the easier the hair removal becomes.
8. Do not surpass the paper band surface.
9. Hold with one hand the paper band tightly, and use the other to press it against the skin.

10. Pull the paper band quickly in the opposite direction the hair growth.
11. After use, apply some smoothing lotion to clean the skin.

## REMARKS AND RECCOMENDATIONS

- Do not pour the wax after treatment if it is still liquid. Turn the machine off first and let the wax cool down. Turn it on again and when the wax starts melting at the bottom, the wax block will be easier to remove.
- Do not turn the machine on when the pot is empty.
- When you add wax, move or clean the machine, turn it off.
- Do not directly touch the pot with your hands. Take it out with the provided tool to avoid getting burnt.
- This device is intended for professional use.
- Use alcohols or other wax removal lotions to clean the machine.
- Turn the machine off before cleaning it.
- Do not move the machine when it is heating or when the wax temperature is too high. The temperature may reach the 115°C while working.
- Do not spill the wax. Clean it up if it gets accidentally spilt, otherwise it will become difficult to remove it.
- If the power cord is damaged, get the component at the manufacturer or at an authorised technical service.
- Do not use it on damaged or sensitive areas of the skin.
- Do not pour the used wax into the outer pot of the machine.

## TECHNICAL SPECIFICATIONS

CAPACITY	750 ML
RATED POWER	380w
RATED VOLTAGE	AC 220V ~ 240V

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